



### POLYURETHANE PAD

#### For cleaning

Use soft cloth and non-abrasive neutral detergent. Do not use alcohol, solvent-based, acid or abrasive cleaners

#### For disinfection

Denatured alcohol or ethyl alcohol in a maximum concentration of 5% is recommended.\*

**Before disinfecting the entire surface of any component, it is advisable to proceed with tests on a small part of the product.**

#### PRODUCT NAME

**Fox**

#### TYPE

**PU pad (Soft)**

#### MATERIALS

**Polyurethane plastic**

\* Analysis of the effects resulting from the application of chemicals on product surfaces, performed by an accredited laboratory.