

**POLYPROPYLENE PAD****For cleaning**

Use soft cloth and non-abrasive neutral detergent. Do not use alcohol, solvent-based, acid or abrasive cleaners

**For disinfection**

Denatured alcohol, sodium hypochlorite (0.1% - 0.5% sol.), ethanol (70% sol.), hydrogen peroxide (0.5% sol.) and benzalkonium chloride can be used.\*

**POLYURETHANE PAD****For cleaning**

Use soft cloth and non-abrasive neutral detergent. Do not use alcohol, solvent-based, acid or abrasive cleaners

**For disinfection**

Denatured alcohol or ethyl alcohol in a maximum concentration of 5% is recommended.\*

## PRODUCT NAME

**Midi**

## TYPE

**PP pad (Hard)**

—

**PU pad (Soft)**

## MATERIALS

**Polypropylene plastic**

—

**Polyurethane plastic**

**Before disinfecting the entire surface of any component, it is advisable to proceed with tests on a small part of the product.**

\* Analysis of the effects resulting from the application of chemicals on product surfaces, performed by an accredited laboratory.