



POLYURETHANE PAD

For cleaning

Use soft cloth and non-abrasive neutral detergent. Do not use alcohol, solvent-based, acid or abrasive cleaners

For disinfection

Denatured alcohol or ethyl alcohol in a maximum concentration of 5% is recommended.*

Before disinfecting the entire surface of any component, it is advisable to proceed with tests on a small part of the product.

PRODUCT NAME

Npr

TYPE

PU pad (Soft)

MATERIALS

Polyurethane plastic

* Analysis of the effects resulting from the application of chemicals on product surfaces, performed by an accredited laboratory.