



NYLON PLASTIC COMPONENTS

For cleaning

Use soft cloth and non-abrasive neutral detergent. Do not use alcohol, solvent-based, acid or abrasive cleaners.

For disinfection

We recommend the use of sodium hypochlorite (0.1% - 0.5% sol.) or hydrogen peroxide.*

Before disinfecting the entire surface of any component, it is advisable to proceed with tests on a small part of the product.

PRODUCT NAME

Adapter plate

TYPE

Seat adapter

MATERIALS

Nylon plastic

* Analysis of the effects resulting from the application of chemicals on product surfaces, performed by an accredited laboratory.