



POLYPROPYLENE PLASTIC COMPONENTS

For cleaning

Use soft cloth and non-abrasive neutral detergent. Do not use alcohol, solvent-based, acid or abrasive cleaners

For disinfection

Denatured alcohol, sodium hypochlorite (0.1% - 0.5% sol.), ethanol (70% sol.), hydrogen peroxide (0.5% sol.) and benzalkonium chloride can be used.*

Before disinfecting the entire surface of any component, it is advisable to proceed with tests on a small part of the product.

PRODUCT NAME

Trasla 2

Trasla 2 Cable

TYPE

Quick connection seat

MATERIALS

Polypropylene plastic

* Analysis of the effects resulting from the application of chemicals on product surfaces, performed by an accredited laboratory.